

FAST ACTING SURFACE DISINFECTION FOAM FOR SENSITIVE SURFACES



Meliseptol® Foam pure ... quick and gentle

PROPERTIES

- Foam for rapid disinfection of non-invasive medical devices and small areas
- Good material compatibility also with alcohol-sensitive materials, such as acrylic glass, ultrasonic probes, examination chairs
- Perfume-free, mild smell
- Active against bacteria (including mycobacteria and MRSA) fungi, enveloped viruses (including HBV, HCV, HIV)¹ and effective against Rota-, Polyoma-, Noro- and Avian Influenza virus
- Dermatological test result: «very good»
- Compatible with B. Braun Wipes dispenser system

1) acc. recommendation of RKI, Bundesgesundheitsbl. 03-2017, DW / RKI suspension test



IN A NUTSHELL

- For alcohol-sensitive materials – also ultrasonic probes
- No aerosol build-up when sprayed
- Perfume-free
- Short exposure time
- Dermatological test result: «very good»
- Also suitable for application in food area
- Listing: the DGHM/VAH list and the IHO virucide list
- Applicable as impregnating liquid with the B. Braun Wipes

APPLICATION INSTRUCTIONS

Fully soak the areas with Meliseptol® Foam pure and rub with a wipe. Keep the surface wet until the exposure time has elapsed. Do not wipe the surface dry. For exposure time, see table.

MATERIAL COMPATIBILITY

To preserve the value of your high tech equipment, e.g.

- Perfusor® Space
- Infusomat® Space
- Dialog+ & Dialog iQ
- Acrylic glass

Spectrum of activity Use under clean conditions at ambient temperature (min.20°C)	Exposure Time
Bactericidal, yeasticidal with mechanical action (EN 13727, EN 13624, EN 16615)	1 min.
Bactericidal, yeasticidal without mechanical action (EN 13727, EN 13624, EN 17387)*	1 min.
Mycobactericidal (EN 14348)	3 min.
Tuberculocidal (EN 14348, EN 14563)	3 min.
Fungicidal (EN 14562, A. brasiliensis)	60 min.
Virucidal against enveloped viruses (EN 14476, DVV/RKI incl. HBV, HCV, HIV)**	1 min.
Avian Influenza-A-Virus (EN 14476, DVV/RKI)	15 sec.
Norovirus (EN 14476, MNV)	3 min.
Rotavirus (DVV/RKI)	1 min.
BVDV (DVV/RKI)	1 min.
Polyomavirus SV40 (DVV/RKI)	5 min.

* the modified test conditions correspond to the requirements of EN 17387

** acc. Recommendation of RKI, Bundesgesundheitsbl. 03-2017, DVV/RKI suspension test

COMPATIBLE WITH THE B. BRAUN WIPES DISPENSER SYSTEM
Meliseptol® Foam pure can be used as impregnating liquid with the B. Braun wipes dispenser system. The solution can remain in the dispenser system for up to 4 weeks.

PRODUCT SIZE	REF
750 ml Foam spray bottle	19796, 19797, 19864
5 l canister	19288, 19289

Physico-chemical data ready-to-use solution

Appearance	clear, colorless liquid
pH-value	approx. 7
Density (20 °C)	approx. 0.98 g/ml
Flash point (DIN 51 755)	+ 31.5 °C

Meliseptol® Foam pure – composition:

100 g solution contains: 17 g Propan-1-ol, 0.23 g Didecyltrimethylammoniumchloride
Excipients: < 5 % non-ionic surfactants, purified water, Ingredients in accordance with the Regulations on Detergents EC 648/2004: < 5 % non-ionic surfactants, Labelling of dangerous goods: see material safety data sheet (MSDS), Cautions: Use disinfectants safely. Always read the label and the product information before use. Do not use the product after the expiry date.



For all sensitive surfaces