



Sterillium® med

The hand disinfectant with virucidal efficacy within hygienic hand disinfection. Featuring the proven Sterillium® skincare complex. Colourant- and fragrance-free.



Image may vary according to local requirements

Application

Sterillium® med is suitable for hygienic and surgical hand disinfection. Sterillium® med is free from colourants and fragrances and thus is particularly suitable for users with sensitive skin.

Thanks to its highly efficient formula, Sterillium® med provides full virucidal activity incl. Norovirus within hygienic hand disinfection and thus is ideally suitable for year round infection prevention.

Directions for use

Hygienic hand disinfection: Apply sufficient amount (min. 3 ml) of undiluted hand disinfectant to the cupped hand so that all areas of the hands can be wetted thoroughly and completely. Carefully rub in hand disinfectant for 30 seconds – ensure to cover all areas of skin.

Surgical hand disinfection: Thoroughly rub Sterillium® med into hands and forearms up to the elbows. When treating hands, pay particular attention to fingertips, palms and thumbs. Hands and forearms are to be kept wet during the entire application time of 1.5 minutes.

Characteristics

- comprehensively efficacy, including virucidal activity with application time for hygienic hand disinfection
- increases skin hydration, preserves skin elasticity and the natural smoothness of the skin
- all-year-product with quick efficacy
- high skin tolerability
- maintains the natural pH value of the skin
- colourant- and fragrance-free
- with proven skin care complex



Proven efficacy	Testing method	Exposure time
Bacteria / Fungi		
Hygienic hand disinfection	EN 1500	30 sec
Surgical hand disinfection	EN 12791	1.5 min
Bactericidal	EN 13727	15 sec
Tuberculocidal	EN 14348	15 sec
Mycobactericidal	EN 14348	15 sec
Fungicidal	EN 13624	30 sec
Viruses		
Virucidal activity against enveloped viruses	EN 14476	15 sec
Limited spectrum virucidal activity	EN 14476, prEN 17430	15 sec
Norovirus	EN 14476	10 sec
Virucidal	EN 14476	15 sec

Composition

Active substances per 100g:
Ethanol 85.0 g.

Microbiology

Bactericidal, fungicidal, tuberculocidal, mycobactericidal, virucidal against enveloped viruses, limited spectrum of virucidal activity incl. Norovirus, virucidal.

Sterillium® med



Chemical-physical data

Appearance clear, colourless
Density (at 20°C) approx. 0.82 g/cm³
Flashpoint 20°C
(acc. to EN ISO 3679)

Stability

In opened, and tightly closed container, with mounted single-use pump or when using the Eurodispenser 1 plus / 1 plus touchless: 12 months.

Expert assistance

HARTMANN SCIENCE CENTER
Tel: +49 (0) 40 - 54 00 6 -111
E-Mail: science-center@hartmann.info



Research for
infection protection
hartmann-science-center.com

Product	Content	Item no.
Sterillium® med	50 ml bottle	on request
	100 ml bottle	on request
	500 ml bottle with pump or without	on request
	1000 ml bottle with pump or without	on request

Please note: that the availability of products in the Sterillium® range may vary in different countries and regions. Contact your local distribution partner for more information. The recommendations regarding our preparations are based on scientific tests and are given in good faith. More detailed recommendations, e.g. regarding material compatibility, are possible only in separate, individual cases. Our recommendations are not binding and do not constitute a guarantee. They do not preclude a company's own testing for the intended purpose and process. In this respect we cannot accept any liability. This is in accordance with our general conditions of sale and supply.

Use disinfectants safely. Always read the label and product information before use.

Please amend in accordance with local requirements (e.g. law of advertising, product status, CLP labelling)



Helps. Cares. Protects.

Manufacturer:
BODE Chemie GmbH
A company of the HARTMANN GROUP
Tel: +49 40 5 40 06-0 · Fax: -200
info@bode-chemie.de
www.bode-chemie.com