

schülke wet wipes dispenser

Filling under hygienic conditions

Step 1

We recommend that you disinfect your hands and the work surface and wear an overall, protective goggles and gloves*. This will protect you and the schülke wipes.

* as required by paragraph 4.1.3 of TRBA 250



Step 2

Open the lid of the wet wipe dispenser and insert the unpacked fleece roll into the dispenser. Pull the inmost wipe a little from the centre.



Step 3

Pour 2 litres of ready-to-use product** or 3 litres of diluted concentrate solution once slowly in a spiral shape over the roll of wipes in the bucket. (schülke wipes 40 / 70 = 1,5 litre).

** antifecht* N liquid, buratorn* rapid, mikroizid* AF liquid, mikroizid* universal liquid, mikroizid* sensitive liquid, perform* advanced alcohol EP, perform* classic alcohol EP, terralin* liquid



Step 4

Now pull the inmost wipe of the roll into the star-shaped dispensing opening of the lid and dispose of it.



Step 5

Place the lid onto the dispenser and close it until it locks into place.

Now carefully close the protective cap of the dispenser.



Step 6

The wipes take 30 minutes to become fully saturated.



Wait
30 minutes



Step 7

In the meantime, complete each charge label supplied with each cloth roll and stick it onto the field provided.



Step 8

The schülke wet wipes dispensing system is now ready for use. All schülke products have a shelf life of 28 days. Except for terralin* PAA with 12 hours of shelf life.



Hygienic refilling

1. Please dispose any remaining product and wipes after the expiry date or when refilling.
2. **Clean** the dispenser and the lid thoroughly with an efficient and absorbent wipe (e.g. mikroizid* sensitive wipes), to remove any soiling.
3. **Disinfect** the lid and the dispenser thoroughly. We recommend using alcoholic disinfectant wipes like the mikroizid* AF wipes.
4. The lid and the dispenser can also be reprocessed for 10 minutes at 70°C or for 1 minute at 90°C.
5. The dispenser and the lid must dry after disinfecting.

The refilling of the dried dispenser must occur in the same way as the initial filling.



Discard solution



Dispose of wipes



Clean, disinfect & dry



Clean & disinfect