



schülke wipes

Wet wipes dispenser for using schülke surface disinfectants in all areas of hospitals, nursing homes and emergency services.



Our Plus

- comprehensively usable
- easy application
- 40, 70, 100 or 111 wipes per roll

Application areas

Wet wipes dispenser to be used in conjunction with schülke surface disinfectants. schülke wipes can be used for disinfecting the surfaces of and cleaning non-invasive medical devices as well as all types of surfaces. We offer the following articles:

- schülke wipes 40 (30 x 30 cm)
- schülke wipes 70 (28 x 23 cm)
- schülke wipes 100 (30 x 30 cm)
- schülke wipes 111 (30 x 24 cm)

Instructions for use

Filling under hygienic conditions

First disinfect hands and wear new protective gloves and a gown. During the filling process, please ensure that the gloves are not contaminated by other activities. Open the lid of the wet wipe dispenser and insert the unpacked fleece roll. Pull the innermost wipe a little out of the middle of the role. Then add 2 litres of ready-to-use product or 3 litres of diluted-concentrate-solution once slowly in a spiral shape over the role (schülke wipes 40/ 70 = 1.5 litres). Only use the products and concentrations recommended by schülke for this purpose. (Please observe the respective product information.)

Pull the inmost wipe of the roll into the star-shaped dispensing opening of the lid and firmly place the lid onto the dispenser. After 30 minutes, the wipes are completely saturated and ready for use.

In the meantime, as for each refilling, record the product name, the selected concentration to be used, the exposure

time and the filling time on the label, which is supplied with each fleece roll. Stick it onto the field provided. For medical products and surfaces that are sensitive to alcohol (e.g. pads, displays, acrylic glass, ultrasound transducers) we recommend you to soak the wipes in our mikrozid sensitive liquid. Remove any visible dirt prior to disinfection. Wipe surfaces thoroughly with a soaked fleece wipe. Make sure to wet surfaces completely and keep them wet for the whole exposure time - do not use wipes that have dried out. The information provided by the manufacturer(s) of the disinfection product(s) used on reactions times, application concentrations, range of action and any specific instructions for use is to be heeded. The reusable schülke wipes are not suitable for use in risk areas. schülke wipes safe & easy bag-in-box/ schülke wipes safe & easy bagless are recommended in this instance. The disinfectant used should also be checked in relation to whether or not it is suitable to be used in risk areas.

Cleaning and disinfection

When using the saturated wipes, make sure the wiped surfaces are completely wetted, and that the dispensing opening is properly closed after each removal of a wipe, to prevent the wipes from drying up. For professional use only. Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Not for final disinfection of semicritical and critical medical devices!

Refilling under hygienic conditions

The dispenser needs to be cleaned and disinfected before refilling it with a new fleece roll.

CE 0297



Special advice

The expiry date for all schülke products is after 28 days, apart from terralin® PAA with 12 hours.
Further information can be found in our leaflet.

Information for order

Item	Delivery form	Item no.
schülke wipes (6x100 wipes)	1/Carton	on request
schülke wipes (6x70 wipes)	1/Carton	on request
schülke wipes (6x111 wipes)	1/Carton	on request
schülke wipes 100/111 dispenser bucket	1/Carton	on request
schülke wipes mini 40/70 dispenser bucket	1/Carton	on request
schülke wipes (10x40 wipes)	1/Carton	on request
schülke wipes 70 dispenser box	1/Carton	on request
schülke wipes 100/111 dispenser box	1/Carton	on request
schülke wipes 130 (6 x 130 wipes)	1/Carton	on request
schülke wipes 70 dispenser box (5x1 pc)	1/Carton	on request
schülke wipes 100/111 dispenser box (5x1 pc)	1/Carton	on request
schülke wipes 100/111 dispenser bucket (5x1 pc)	1/Carton	on request
schülke wipes mini 40/70 dispenser bucket (5x1 pc)	1/Carton	on request
schülke wipes Box Deckel 70/100/111	1/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
Wall bracket schülke wipes box	134435
Wall bracket schülke wipes bucket	134430

Related Products

- antifect® N liquid
- buraton® rapid
- mikrozyd® AF liquid
- mikrozyd® sensitive liquid
- perform® advanced alcohol EP
- perform® classic alcohol EP
- perform® classic concentrate Q-Plus
- terralin® liquid
- terralin® PAA
- terralin® protect

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: info@schuelke.com

terralin® PAA can only be used in combination with the schülke wipes 40 and 3L solution. The expiry date is after 12 hours.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com