

Actim[®] Fecal Blood

THE CLEAN AND SIMPLE WAY TO FIND OCCULT BLOOD IN FECES

Actim Fecal Blood is a high-quality, user-friendly rapid test to detect occult blood in feces. Its advanced and unique design offers a very practical, hygienic solution that requires no dietary restrictions.

The test quickly identifies patients who may have severe gastrointestinal diseases and need further medical care – in just 10 minutes. And thanks to our easy sampling method, patients are more willing to do the test, as it comes without any fear of uncomfortable exams.

With millions of Actim Fecal Blood tests used, it's clear that both its quality and ease-of-use are widely valued.



 www.medixbiochemica.com



ACTIM FECAL BLOOD - SIMPLE SOLUTION FOR BOTH PATIENTS AND PROFESSIONALS

Our Actim Fecal Blood test is ideal in every way: it's simple, quick, accurate, and safe to use.

Sample collection can be done anytime – even at home, and is as comfortable as possible. The sample device is small enough for mailing back to lab, and the specimen can be kept at room temperature even for a full week before testing.

Our one-step test is effortless – taking just 15 seconds of active lab time. It's always ready to use since it requires no instrument, can be stored at room temperature and has everything needed for testing. What's more, it's very practical when doing several tests at once – or even for mass screenings.

LEAK-PROOF SAMPLING DEVICE

Our innovative solution offers exceptionally hygienic testing and sample handling. The sampling device is closed and there is no need to open the tube for testing – which makes it safe and worry-free.

The needed sample size is minimal, and the sampling tube even adjusts the amount of feces needed for testing – ensuring that your test results are always accurate.

HOW IT WORKS

Actim Fecal Blood test is intended to detect bleeding in the gastrointestinal tract even when there are no visible signs. Our test is of premium quality – with a specificity of 94 % and optimal sensitivity, which helps avoid unnecessary, uncomfortable and costly exams.

The test is immunological and based on monoclonal antibodies specific only against human hemoglobin – and there are no cross reactions with other hemoglobins. In other words, no dietary restrictions concerning red meat, and vegetables or fruits including peroxidase activity are required – so testing can be done immediately.

Early diagnosis is vital

Feces don't normally contain measurable amounts of blood, thus occult blood in feces is a sign of bleeding in the gastrointestinal tract. This may be caused by, for example, hemorrhoids, but may also be a sign of colorectal cancer. Diagnosing occult bleeding rapidly is crucial and may even save lives!



1. Perforation
2. Test activation
3. Result interpretation



CONTACT US

Medix Biochemica
Klovinpellontie 3, FI-02180 Espoo, Finland
medix@medixbiochemica.com

Ordering information	
Actim Fecal Blood 20 tests kit	30331ETAC
Actim Fecal Blood 10 dipsticks	30361ETAC
Actim Fecal Blood 20 buffers	30372ETAC
Actim Fecal Blood Controls	30300ETAC