Aesculap

B. Braun VetCare

- GB** Translation of the original instructions
Exacta/Isis
- D** Originalbetriebsanleitung
Exacta/Isis
- F** Traduction de la notice originale
Exacta/Isis
- E** Traducción del manual original
Exacta/Isis
- I** Traduzione delle istruzioni originali
Exacta/Isis
- NL** Vertaling van de oorspronkelijke gebruiksaanwijzing
Exacta/Isis
- S** Översättning av bruksanvisning i original
Exacta/Isis
- RUS** Инструкция по применению
Exacta/Isis
- PL** Tłumaczenie instrukcji oryginalnej
Exacta/Isis
- TW** 使用说明书
Exacta/Isis

B | BRAUN
SHARING EXPERTISE

English	4
Translation of the original instructions	
Deutsch	22
Originalbetriebsanleitung	
Français	40
Traduction de la notice originale	
Español	58
Traducción del manual original	
Italiano	76
Traduzione delle istruzioni originali	
Nederlands	94
Vertaling van de oorspronkelijke gebruiksaanwijzing	
Svenska	112
Översättning av bruksanvisning i original	
Русский	130
Инструкция по применению	
Polski	150
Tłumaczenie instrukcji oryginalnej	
國語	164
娑鈔壁讨鈕	

Contents

1.	Names of parts.....	7
2.	Safety	7
2.1	Explanation of warnings	7
2.2	Intended use.....	8
2.3	Important information about dangers	8
13.1	General safety information	9
2.4	Safety during operation.....	11
3.	Preparation.....	13
3.1	Charging	13
3.2	Storing the power cable	14
4.	Operation	14
4.1	Switching On and Off	14
4.2	Comb attachment	14
5.	Cleaning and care	15
6.	What should I do if	17
7.	Disposal.....	18
8.	Technical data	19
9.	Two-year guarantee.....	20

Thank you

for choosing the **Exacta/Isis**, the professional trimmer from Aesculap.

It utilises state-of-the-art technology, is innovative and has a strong emphasis on design. When developing the trimmer, the requirements for a professional trimmer used in a hair salon were taken into consideration.

The trimmer is specifically designed for precision cutting of hair contours.

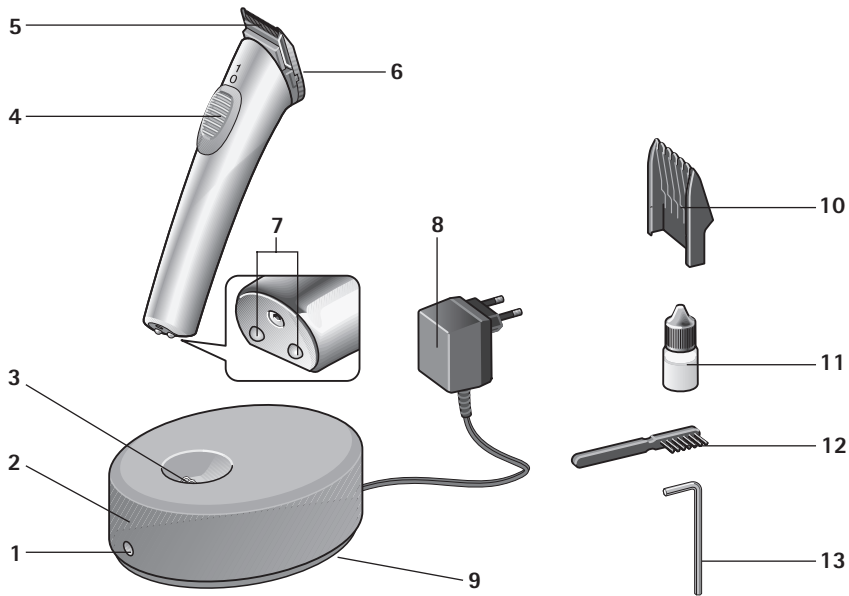
In addition to optimum cutting performance, the **Exacta/Isis** provides the following advantages:

- Light weight
- Quiet, low-vibration motor
- Battery operation
- Rapid charge in one hour
- 4-stage cutting length adjustment with comb attachment
- Cable storage compartment

We hope you enjoy working with the trimmer.

Best wishes

Aesculap Suhl GmbH

1. Names of parts

- 1 Charging indicator
- 2 Charging unit
- 3 Charging contacts
- 4 On/Off switch
- 5 Blade set
- 6 Blade set fixing screw
- 7 Charging contacts of the appliance
- 8 Mains adapter
- 9 Cable storage compartment
- 10 Comb attachment
- 11 Oil bottle
- 12 Cleaning brush
- 13 Allen key (in base of charging unit)

2. Safety

Please read the instructions carefully, especially the chapter on "Safety", **before** using the trimmer. This will protect you from possible dangers and the appliance from damage due to incorrect operation.

Keep these instructions in a safe place for future reference. If you pass on the appliance to someone else, be sure to hand over these instructions as well.

2.1 Explanation of warnings

In these instructions you will find the following warnings:

DANGER

High risk. Failure to observe the warning can cause serious injury.

WARNING

Medium risk. Failure to observe the warning may cause damage to property.

NOTE

Low risk. Points that you should observe when using or handling the appliance.

2.2 Intended use

- The trimmer must only be used for cutting hair of small animals.
- The appliance is designed for indoor use only. Do not use outdoors (garden, patio etc.).
- The appliance was developed for professional use in grooming-salons and veterinary practice. Provided the safety information is observed, it can also be used in the home.

2.3 Important information about dangers**Danger of electric shock!**

- The appliance must never be used or charged in the bath, shower, over a full wash basin or other water container, or with wet hands.

- Keep the appliance, charging unit and cord away from moisture, water and other liquids, and do not operate with wet hands.

- Do not immerse the appliance, charging unit or cord in water or rinse them off under running water.
- If the appliance has fallen into water, it must not be used after that.
- Additional protection is afforded by installing a **residual current device** with a rated tripping current of less than 30 mA in your home electrical system. We recommend you get an electrician to install a residual current protection system to protect the circuits..

**WARNING**

The shaver head can become hot after extended use!

NOTE

The Clipper may be operated only with a power supply according to the specifications on the rating plate.

13.1 General safety information

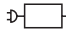
**DANGER**

- Never use the appliance if:
 - the appliance, charging unit or cord are damaged.
 - the blade set is damaged or breaking up, as this could cause injuries to the skin.
 - the appliance has fallen or been dropped, shows signs of damage or is not functioning properly.
 - the appliance is so badly damaged that electrical parts are exposed.

In all such cases, disconnect from the mains immediately and inform Technical Customer Service.

- Check whether the mains voltage agrees with the voltage specified on the rating plate.

- Only operate device with the original power supply.

If the device has the marking  T10, any T10 model power supply can be used.

This device can be used by **children** aged 8 or older and people with impaired physical, sensory or mental capacity or lack of experience and knowledge if they are supervised or have been instructed regarding safe use of the device and understand the resulting risks. **Children** may never play with the device. Cleaning and **user maintenance** may not be performed by **children** without supervision.

- Children should be supervised to ensure that they do not play with the appliance. Be careful to prevent children from getting injured by touching the blade set. Keep the appliance out of reach of children.
- Also keep the packaging material, e.g. plastic film, away from children.
- Lay the power cable so that nobody can trip over or tread on it.
- Protect the power cable from damage. Never pull on the power cable. When pulling the mains adapter out of the power socket, always grasp the adapter itself. Keep the cable away from hot surfaces. Check the power cable regularly for damage.
- Do not expose the appliance to temperatures below 0 °C or above +40 °C for any length of time. Avoid exposure to direct sunlight.
- To avoid risks, repairs must only be carried out by authorised Aesculap Customer Service Centres. This also applies to replacing the rechargeable battery.

- If the cord to the appliance is damaged, it must be replaced by the manufacturer or the manufacturer's customer service or similarly qualified persons in order to avoid risks.
- Unplug device from the power supply for maintenance and part replacement.
- The blade set must not be dismantled.

2.4 Safety during operation

DANGER

Do not touch the blade set of the active appliance.

- To ensure proper functioning of the appliance, the blade set must be cleaned regularly and oiled in the appropriate places (see "Cleaning and care" on page 15).
- The appliance must be switched off whenever:
 - you put it down, e.g. to recharge it or stop cutting hair for a short period.
 - you fit or remove the comb attachment.
 - you want to clean or oil the appliance.
- Use the trimmer only in combination with the original Aesculap accessories (charging unit, comb attachment etc.).
- Never attempt to remove dust or foreign bodies from inside the appliance with pointed objects (e.g. with the tail of a comb).

NOTE

The safety of Aesculap electrical appliances is in line with the recognised state of the art and the German Equipment Safety Act.

Light-emitting devices such as LEDs or infrared light do not exceed Laser class 1 of IEC 825-1.

Given the present state of the art, components of electrical appliances are subject to a certain degree of wear over time. To ensure perfect functioning and safety, we therefore recommend regular inspection by the Technical Customer Service of Aesculap Suhl GmbH.

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Further service addresses you can demand from the mentioned above address.

3. Preparation

3.1 Charging

The appliance as supplied is not yet ready for operation. Before plugging in the trimmer, make sure that the mains voltage agrees with the voltage specified on the rating plate.

Prior to first use, the appliance should be charged for 2 hours.

It is not possible to "overcharge" the appliance.

NOTE

Should you use both the new hair trimmer and the previous model, you can charge the new appliance in the old charging unit (charging time: 16 hours) and/or the old appliance in the new charging unit (charging time: 16 hours).

- Insert the mains adapter **8** in a power socket.
- Place the charging unit **2** on a flat surface.
- Place the appliance in the charging unit. It positions itself and starts charging. The red charging indicator **1** is always on when the appliance is charging. Make sure the On/Off switch **4** is in the 0 position when charging.

To maintain the charging capacity of the battery we recommend occasionally draining the battery until the motor stops.

3.2 Storing the power cable

If you do not need the full length of the cord, part of it can be stored in the cable storage compartment **9**.

- Pull out the mains plug.
- Turn the charging unit upside down.
- Wind the excess cable round the holder in the base, then run the cable through the small recess in the housing to the outside.

4. Operation

4.1 Switching On and Off

- To switch the appliance on, slide the On/Off switch **4** forward.
- To switch the appliance off, slide the On/Off switch to position **0**.

4.2 Comb attachment

- Switch off the trimmer and slide the comb attachment onto the blade set until it snaps into place. You can now start trimming hair with the trimmer in the usual way.

- The comb attachment can be adjusted to 4 different cutting lengths (approx. 3 to 6 mm). To do so, push it to the required position on the blade set.

The further you push the comb attachment onto the blade set, the shorter the cutting length.

5. Cleaning and care

To ensure proper functioning and full performance from your trimmer, clean and oil the appliance regularly.

Cleaning the blade set

Damage to the clipper head caused by its removal!

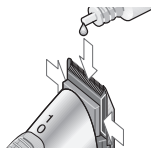
- Never remove the clipper head. Once removed it cannot be reattached.

If used regularly, the blade set **5** should be thoroughly cleaned **once a day**.

- Switch the appliance off.
- Remove the comb attachment if fitted.
- Use the cleaning brush **12** to remove any hairs.

Oiling the blade set

To maintain the high-quality cutting performance of the blade set as long as possible, it must be oiled regularly – preferably daily.

- Remove the comb attachment if fitted.
- Lubricate the areas where there is contact between the upper and lower parts of the blade set (see illustration).

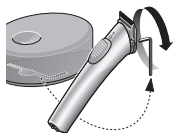
Replacing the blade set

The blade set is subject to wear. Even if you look after the appliance properly, the cutting performance will deteriorate. Replace the blade set when it is worn. The blade set can be ordered separately as a spare part from Aesculap Customer Service (see addresses at the end of these instructions).

Damage to casing when screwing on clipper head!

- Do not over tighten the screw.

- Make sure the On/Off switch **4** is in the 0 position.
- Remove the Allen key **13** from the base of the charging unit and use it to loosen the blade set screw **6**.
- Loosen the screw fully and remove it.

- Push the blade set upwards with your thumb.
- Push the new blade set on until it snaps into place and secure it using the screw.

While the blade set is removed, you can use the cleaning brush to remove residual hair from inside the appliance.

Cleaning the appliance

- Clean your appliance occasionally with a damp cloth, then wipe it dry.

Cleaning the charging contacts

If the charging contacts **7** on the appliance or the charging contacts **3** on the charging unit are dirty, charging may not be possible or may be interrupted.

- Clean the contacts using a cloth or wad of cotton wool soaked in alcohol or spirit.

Care of the NiMH rechargeable battery

To maintain the charging capacity of the battery we recommend occasionally draining the battery until the motor stops.

If the battery is run down (motor stops), be sure to switch the On/Off switch to 0 and place the appliance in the charging unit.

- Make sure you do not leave the appliance switched on when putting it aside if the battery is run down.
- Make sure the appliance is switched off (On/ Off switch to 0) when you place it in the charging unit.

6. What should I do if ...

... the appliance will not switch on?

- Is the battery low? Plug in the mains adapter for the charging unit and charge the trimmer.
- Is the blade set dry? The blade set must be oiled. Proceed as described on page 15.

... the blade set "sticks"?

- The blade set must be oiled. Proceed as described on page 15.

... the charging indicator does not light up when charging?

- Clean the charging contacts.
- The appliance may not be positioned correctly in the charging unit. Position it correctly.
- Is the charging unit properly connected?

... the appliance switches itself off?

- The battery is low. Charge the trimmer.

... the battery will no longer charge?

- Send the appliance to Aesculap. Our customer service technicians will fit a new battery and ensure environmentally sound disposal of the old one.

... the device does not turn on?

- Is the shaver head dirty/sticky? Unscrew the shaver head.
- Check that the machine is running.
 - If yes, clean and oil the shaver head, see "Cleaning and care" from page 15.
 - If no, send the machine to the Aesculap repair service.

7. Disposal

This electric appliance is fitted with an integral battery.

NOTE

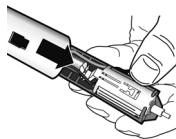
The battery must be removed prior to disposing of the appliance!


- Unscrew the shearing head.
- Lever off the control button (see illustration).

- Push out the inner part (see illustration).

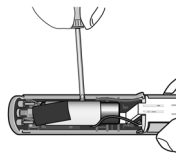- Pry out the battery (see illustration).
- Dispose of the battery, separately.

At the end of its service life, have the electric appliance returned to the supplier/stockist or the appropriate collection point at the local, public waste disposal facility (recycling center, materials collection center, etc.).

Electric appliances must not be disposed of as residual or general waste.

This also applies to electric appliances ultimately used for other than private purposes, e.g. for trade or commerce. Should the relevant local, public disposal facilities refuse acceptance of electric devices used for other than private purposes, the facilities' recycling advice office will provide information pertaining to the environmentally compatible disposal of the items in question.

8. Technical data

Product	Exacta/Isis
Mains adapter	
Input voltage	220-240 V ~ 50/60 Hz
Emission sound pressure level	LpA ≤ 70 dB(A)
Total vibration level	ah = 2.76 m/s ²
Attachment	1 comb attachment
Dimensions	140 x 30 x 36 mm
Weight	120 g

Subject to technical change without notice.

9. Two-year guarantee

Dear Customer,

Thank you for choosing a product from our company.

For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices, using high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Parts that are typically subject to wear, and for which we are therefore not liable, are the battery and clipper head. Defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

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