# GRIPIT®

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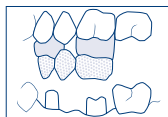
Abb. 4

# GRIPIT®

## Bite-cushion for the removal of crowns, bridges and inlays

### Instructions for use


Gripit: Bite-cushions for the removal of crowns, bridges and inlays without the help of any instruments. Ideal for the removal of temporary teeth particularly when prepared dentures must be removed after trying in. It is very annoying when prepared dentures get damaged by instruments. Gripit removes the dentures easily, safely and without causing damage!

**Instructions for use:** Prior to use Gripit must be heated in its foil in hot water. Let the patient bite down with even pressure for approx. 10 seconds (**pic. 1**). Then ask the patient to open his mouth in a **jerky movement** (**pic. 2**). In case several teeth are concerned (bridges), the use of two Gripit is recommended (**pic. 3 + 4**).


No reason to worry in case a Gripit is swallowed accidentally: Gripit is non-toxic and digestible. Gripit is water-soluble and possible residues can be removed without any difficulties with an instrument.

**Note:** Gripit has a very strong adhesive force and must only be used if the antagonist is firm and in healthy condition.


Gripit is a disposable product and its re-use leads to a high risk of infection. Furthermore, the product functionality can no longer be guaranteed if used more than once.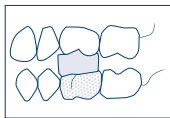


pic. 1

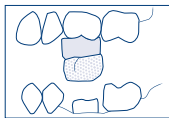pic. 2

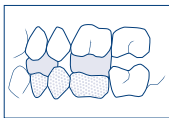pic. 3

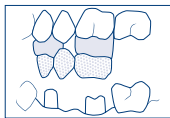pic. 4