# CRYO PROFESSIONAL

**INSTRUCTIONS FOR USE**

EN

**MODE D'EMPLOI**

FR

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DE

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NL

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ES

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PT

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IT

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PL

**BRUKSANVISNING**

NO

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FI

**BRUKSANVISNING**

SE

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CN

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HK

**PETUNJUK PENGGUNAAN**

IN

Read instruction for use carefully prior to using this product  
Use of the product in accordance with its instructions at all time  
Using the product in a way other than instructed, may lead to thermal reactions,  
skin damage, nerve damage and scar formation  
For external use only  
For use by medical professionals only

EN

## CONTENTS

1x Aerosol dispenser: Filled with 170 ml cryogen comprised of dimethyl-ether with propane & isobutane. The cryogen does not damage the ozone layer.  
1x Box foam-applicator: The package includes 2mm small foam-applicators and/or 5mm medium foam-applicators.  
For single use per patient only.  
1x Instructions for use.

## DESCRIPTION

The product is used in the treatment of various benign epidermal skin lesions. Different sizes of foam-applicators supplied are to be used depending on the treatment of the lesion. Different types of skin lesions are treated with different freezing times.

***WARNING: Do not use other brands of foam-applicators. Use of other brands of foam-applicators will affect the performance of the product and could lead to unwanted damage to the skin and underlying tissues.***

## INTENDED USE / INDICATIONS

The product is intended for use on patients older than 4 years of age with skin lesions.  
The product is indicated for treatment of the following skin lesions:

- Genital warts
- Molluscum Contagiosum
- Acrochordon (Skin tags)
- Verruca (Plantaris, Vulgaris, Plana)
- Seborrhic Keratosis (SKs)
- Actinic Keratosis (AKs, facial and non-facial)
- Lentigo's (facial and non-facial)

## CONTRAINDICATIONS

Do not use Cryo Professional:

- When there is or has been a confirmed diagnosis of cancer and treatment for cancer
- For diabetics and persons with circulatory problems
- At patients with cryoglobulinaemia
- At patients using immunosuppressant medicines

## WARNINGS & PRECAUTIONS

- Do not use in the vicinity of other medical or liquid gases
- Do not use in the vicinity of or with diathermy equipment
- Hypopigmentation and / or inflammatory hyperpigmentation may occur in highly pigmented and lightly pigmented skin
- Deep-freezing in regions of peripheral arteries such as in the extremities may result in necrosis distal to the frozen lesions
- When the dermis is not damaged by the cyrosurgery, scar formation is rare
- Do not saturate the foam-applicator again during use. This will diminish the reservoir function of the foam-applicator
- Do not activate the aerosol while the foam-applicator is in contact with the patient
- Extremely flammable aerosol
- Pressurized container: May burst if heated
- Keep out of reach of children. Do not inhale spray
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources
- No smoking
- Do not pierce or burn, even after use
- Do not spray onto an open flame or other ignition source
- Protect from sunlight
- Do not expose to temperatures exceeding 50 °C/122 °F
- Dispose of contents/container in accordance with local regulations

## INSTRUCTIONS FOR USE

### Skin lesion type

### Recommended freezing times

|  |            |
|--|------------|
| Ano-genital warts (Condyloma acuminatum) | 40 seconds |
| Molluscum Contagiosum                    | 20 seconds |
| Acrochordon (Skin tags)                  | 40 seconds |
| Verruca plantaris                        | 40 seconds |
| Verruca vulgaris                         | 40 seconds |
| Verruca plana                            | 20 seconds |
| Seborrheic keratosis (SKs)               | 40 seconds |
| Actinic keratosis (AKs facial)           | 15 seconds |
| Actinic keratosis (AKs non-facial)       | 40 seconds |
| Lentigo (facial)                         | 15 seconds |
| Lentigo (non-facial)                     | 40 seconds |

1. Determine what kind of skin lesion you are going to treat. Treat large lesions > 5mm in stages.
2. Apply the appropriate antiseptic treatment in order to avoid the risk of infection.
3. Determine the best way to treat the skin lesion. In case of warts, a keratolytic agent may need to be applied prior to the cryotherapy treatment; this will not affect the product performance.
4. Determine which foam-applicator is to be used (2mm or 5mm) and attach it into the dispenser.
5. Remove the protective cap from the aerosol and activate the dispenser with the thumb or fore-finger to saturate the foam-applicator with cryogen until the droplets fall from the foam-applicator. Hold the aerosol upright at all times when saturating the foam-applicator. Keep the aerosol away from persons in the room.
6. Hold the aerosol to a 90° angle so that the foam-applicator is pointing vertically downwards.
7. Wait for 15 seconds for the foam-applicator to achieve its effective temperature.
8. Place the foam-applicator on the skin lesion to be treated exerting a slight pressure. Apply the foam-applicator to the lesion for the recommended treatment time as given in the schedule. During treatment hold the foam-applicator vertically downwards (90° angle) at all times.
9. Repeat the treatment at 2-week intervals for up to 4 times.

### POTENTIAL ADVERSE REACTIONS

Ensure that the patient is informed about potential side effects. The main side effects include:

1. A pain or stinging sensation.
2. Blistering.
3. Hypopigmentation.
4. Inflammatory hyperpigmentation.
5. Skin Atrophy.

### COMPLAINTS AND CUSTOMER SERVICE SUPPORT

Contact the manufacturer or supplier with questions or to report an adverse reaction.

### REFERENCES

Caballero Martinez F, et al. Cutaneous cryosurgery in family medicine: dimethyl-ether-propane spray vs liquid nitrogen. *Aten Primaria*. (1996); 18(5): 211-216

Graells J, Espinola A, Barrio C, Muñoz MD, Román A, Parellada N. Minor cutaneous ambulatory surgery and cryotherapy. Comparative study between a dermatologist and family physicians. *Actas Dermosifiliogr*. 2007;98(3):171-7

Thai KE, et al. Cryosurgery for benign skin lesions. *Australia J Dermatol* (1999); 40(4): 175-184

Thai KE, Fergin P, Freeman M, Vinciullo C, Francis D, Spelman L, Murrell D, Anderson C, Weightman W, Reid C, Watson A, Foley P. A prospective study of the use of cryosurgery for the treatment of actinic keratoses. *Int J Dermatol*. 2004; 43(9):687-692

Wetmore SJ. Cryosurgery for common skin lesions: Treatment in family physician's offices. *Can Fam Physician*. (1999); 45: 964-974

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### REFERÊNCIAS

Caballero Martinez F, et.al. Cutaneous cryosurgery in family medicine: dimethyl-ether-propane spray vs liquid nitrogen. Aten Primaria. (1996); 18(5): 211-216

Graells J, Espinola A, Barrio C, Muñoz MD, Román A, Parellada N. Minor cutaneous ambulatory surgery and cryotherapy. Comparative study between a dermatologist and family physicians. Actas Dermosifiliogr. 2007;98(3):171-7

Thai KE, et.al. Cryosurgery for benign skin lesions. Australia J Dermatol (1999); 40(4): 175-184

Thai KE, Fergin P, Freeman M, Vinciullo C, Francis D, Spelman L, Murrell D, Anderson C, Weightman W, Reid C, Watson A, Foley P. A prospective study of the use of cryosurgery for the treatment of actinic keratoses. Int J Dermatol. 2004; 43(9):687-692

Wetmore SJ. Cryosurgery for common skin lesions: Treatment in family physician's offices. Can Fam Physician. (1999); 45: 964-974

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Caballero Martinez F, et.al. Cutaneous cryosurgery in family medicine: dimethyl-ether-propane spray vs liquid nitrogen.

Aten Primaria. (1996): 18(5): 211-216

Graells J, Espinola A, Barrio C, Muñoz MD, Román A, Parellada N. Minor cutaneous ambulatory surgery and cryotherapy. Comparative study between a dermatologist and family physicians.

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Wetmore SJ. Cryosurgery for common skin lesions: Treatment in family physician' s offices. Can Fam Physician. (1999); 45: 964-974

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